EN. DERMAL CURETTE STERILE

GMDN Code: 63259

Sizes: 2mm, 3mm, 4mm, 5mm, 7mm

Stainless steel, ABS plastic/ LDPE

Sterilized by gamma radiation of minimum 25 kGy and not more than 32 kGy (2.5 M Rads to 3.2 M Rads)

INSTRUCTIONS FOR USE

INTENDED USE

The device is used for scraping of the lesions from the surface of a patient's skin. The device is classified as surgically invasive and for transient use.

MODE OF ACTION

A dermal curette is a single ended medical instrument.

Its tip is rounded and sharp. The curette is held pressing firmly against the skin and through the steady movement the die skin or lesion is removed. The desired thickness can be obtained by adjusting of the angle depth of the curette. The curette should be used immediately after the pouch opening.

INDICATIONS

Different sizes for different thickness available.

Removal of skin from operating area for test specimen. The curette is extremely sharp and thus gives precise and gentle tissue removal, which also facilitates its deployment in the treatment of superficial malign tumours.

CONTRAINDICATIONS

Reused device can transmit contagious diseases to patient and/or user. Adverse event may happen if device is used after expiry date, as expiry date means the latest day of guaranteed sterility. Do not use in cardiac / ophthalmic surgery.

INSTRUCTIONS

- Select proper type and size of dermal durette as per skin area and size of specimen.
- Inspect the package to check that the shelf life is valid and remove a pouch from the package. Peel off the pouch and remove the device.
- Clean the selected area of the skin with a suitable disinfectant.
- To do this, the curette is held between the thumb, index and middle finger, while the skin should be stretched with the fingers of the other hand.
- This holding method enables precise guiding of the device, so that the piece of tissue can be removed in one well-targeted incision.

WARNING

Read the instruction for use.

The product should be used only by a qualified surgeon, doctor or paramedic.

Before use, always check the integrity of the product and packing, along with expiry date. For single use only. Reused device can transmit contagious diseases to patient and/or user. Use de product immediately after opening the pack. Once the pouch is opened, use the device immediately.

The manufacturer is not responsible for any possible consequences because of an improper use of the device. Neither holds responsibility if the device is resterilized or reused.

Sterility of the device is not guaranteed in case that the pouch is broken or torn. Resterilization or reuse provoke changes in the used material and mechanical properties. The intended use can also be altered, as blade sharpness may be blunt.

Keep out of the reach of children.

Used devices should be handled according to the country bio-waste rules and laws.

PRECAUTIONS

Always open the pouch tearing from the indicated symbol.

Devices are extremely sharp, use care while handling. Care must be taken so that the pouch is not opened in an unsterile area otherwise the device which has already been sterilized by gamma radiation will become unsterile. Proper procedures must be used as applicable for handling any sterile product.

To prevent any injury or accident, handle with care during the use of the device. Also do so during the disposal, to avoid any contact or injury due to its sharp nature. In case that performance of the device does not fulfil the intended use, replace it with new one.

KNOWN CHARACTERISTICS OF DEVICE IN CASE OF REUSE

Difficulty to scrap lesions from the surface of the patient's skin.

Risk to transmit any infectious disease.

STORAGE CONDITION

- Keep away from direct sun light.
- Keep away from rain.
- Storage temperature should be 10 to 40°C.
- Humidity of storage area should be 35% RH to 65 % RH.

- Keep away from children.
- Store in cool and dry place.

DISPOSAL SYSTEM

Discard the device in proper waste container and dispose off the product in accordance with the accepted medical practice and applicable local, state and country laws and regulations for handling of bio-medical waste.

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TELIC France SAS
EC REP MDSS GmbH, Schiffgraben 41,