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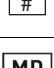
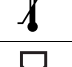
## EN. DISPOSABLE STERILE TUNGSTEN ELECTRODE IMPORTANT INFORMATION

Please carefully read these instructions before use.

### DEVICE DESCRIPTION

The product consists on the active electrode to be connected into the tip of the electrosurgical pencil. Different references are available, with different lengths, tip shapes and coatings. BLAYCO Electrosurgical Accessories can be guaranteed as compatible with all models of BLAYCO Electrosurgical pencils. The maximum applicable voltage of this product is 2.9kVpp.

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## PRECAUTIONS

- Before use, carefully read the instructions for use of the electrosurgical pencil.
- Do not use if the product is damaged or the package is open. Examine electrode tip for defects prior to use.
- Check the expiry date before use.
- Check that electrode length, connection and anchorage are correct after each electrode change.
- Active tip assembly or substitution must be always performed with the electrosurgical pencil disconnected from the electrosurgical unit.
- This device is for single patient use only. It is not intended for disinfection and subsequent reuse, which may result in device failure and/or create the risk of contamination. Do not re-sterilize.

## INSTRUCTIONS FOR USE

1. BLAYCO Electrosurgical Accessories are designed for use with BLAYCO Electrosurgical Pencils. Prior to use, carefully read the manufacturer instructions on how to operate the electrosurgical unit and accessories.
2. Visually examine the package for damages before opening. Replace the product in case that the package or the electrode tip itself shows signs of damage.
3. Check the production and expiry date printed on the package. In case that the product is expired, replace by a backup device within expire date.
4. Open the package using aseptic technique.
5. Operate according to electrosurgical pencil instructions and usual medical protocols.

## STORAGE CONDITIONS

Temperature requirement in warehouse: -20°C to 60°C / -4°F to 120°F

Humidity requirement in warehouse: 10 - 90 % RH.

Atmospheric pressure requirement in warehouse: 500 - 1060 hPa

## SHELF LIFE

The shelf-life of BLAYCO Electrosurgical Accessories can be guaranteed as 3 years.

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