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### FR. ACCESSOIRES ELECTROCHIRURGICAUX

### INFORMATIONS IMPORTANTES

The shelf-life of BLAYCO ElectroSurgical Accessories can be guaranteed as 5 years.

### SHELF LIFE

Humidity requirement in warehouse: 10 - 90 % RH. Atmospheric pressure requirement in warehouse: 500 - 1060 hPa

Temperature requirement in warehouse: -20°C to 60°C / -4°F to 120°F

### STORAGE CONDITIONS

- BLAYCO ElectroSurgical Accessories are designed for use with BLAYCO ElectroSurgical Pencils. Prior to use, carefully read the manufacturer instructions on how to operate the electroSurgical unit and accessories.
- Visually examine the package for damages before opening. Replace the product in case that the package or the electrode tip itself shows signs of damage.
- Check the production and expiry date printed on the package. In case that the product is expired, replace by a backup device within expire date.
- Operate according to electroSurgical pencil instructions and usual medical protocols.

### INSTRUCTIONS FOR USE

- Before use, carefully read the instructions for use of the electroSurgical pencil.
- Do not use if the product is damaged or the package is open. Examine electrode tip for defects prior to use.
- Check the expiry date before use.
- Check that electrode length, connection and anchorage are correct after each electrode change.
- Active tip assembly or substitution must be always performed with the electroSurgical pencil disconnected from the electroSurgical unit.
- This device is for single patient use only. It is not intended for disinfection and subsequent reuse, which may result in device failure and/or create the risk of contamination. Do not re-sterilize.

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